

Fry Laboratories, LLC – Shipping Instructions / List of Contents

Kit Shipping Instructions:

Please Note: The required blood tubes for each test are listed directly above the test on the requisition form. Please make sure that the correct blood tubes are included for test required. **Tubes must be labeled with the patient's name and pre-expiration for us to accept and process the samples.** Please consult the List of Contents below to ensure all of the Kit materials have been provided. You can download a copy of this form for your own records at: www.frylabs.com under the Forms/Orders section. This kit is manufactured by ThermoSafe and is IATA, DOT, and USPS compliant for "Category B UN3373".

1. Freeze the small gel pack that is included with the kit and place it inside the Styrofoam box.
2. Blood may be drawn at any blood draw facility (LabCorp, Sonora Quest, your local doctor's phlebotomist). Some phlebotomists will perform steps 3 to 8 for you.
3. Place the small gel pack, the blood tubes, and the enclosed absorbent paper inside of the Styrofoam box.
4. Use the included red tape to seal the box by taping around the seam between the two Styrofoam box halves.
5. Place the Styrofoam box inside of the enclosed clear Ziploc bag.
6. Place the Styrofoam box inside of the provided white cardboard box along with this sheet. If a UN3373 label is not already stuck to the white box please place it on one of the ends of the white box.
7. Place the box and the completed requisition (**filled out completely or it may result in the delay of testing**) inside the FedEx shipping bag and seal the bag shut. Please look at the billable stamp on the front of the enclosed clinical pack. If it says *2-day*, you may ship your sample any day of the week (**EXCEPT Thursday or Friday**). If it says *Standard Overnight*, you may ship your sample any day of the week (**EXCEPT Friday or Saturday**).
8. Take the package to your nearest FedEx location and they will help you from there. Go to <http://fedex.com/us/> or call 1.800.463.3339 to determine the nearest FedEx location or for assistance with your shipping needs.
9. Results will be available from your physician within 10 business days for most tests and results.

List of Contents:

This item list must be placed between the secondary receptacle(s) and outer packaging in accordance with IATA Packing Instruction 650.

<input checked="" type="checkbox"/> Diagnostic Specimens (Human-Blood)	<input checked="" type="checkbox"/> Absorbent Pad(s)
<input checked="" type="checkbox"/> Requisition	<input checked="" type="checkbox"/> Instructions/List of Contents
<input checked="" type="checkbox"/> Glass Tube(s)	<input checked="" type="checkbox"/> Styrofoam Container
<input checked="" type="checkbox"/> Gel Pack/Refrigerant	<input checked="" type="checkbox"/> Waterproof Tape
<input checked="" type="checkbox"/> Clear Ziploc Bag	